


I/O Ports

The Battery-Powered Medical Grade Monitor has a comprehensive set of I/O ports. The following ports are located along the lower rear edge of the unit.

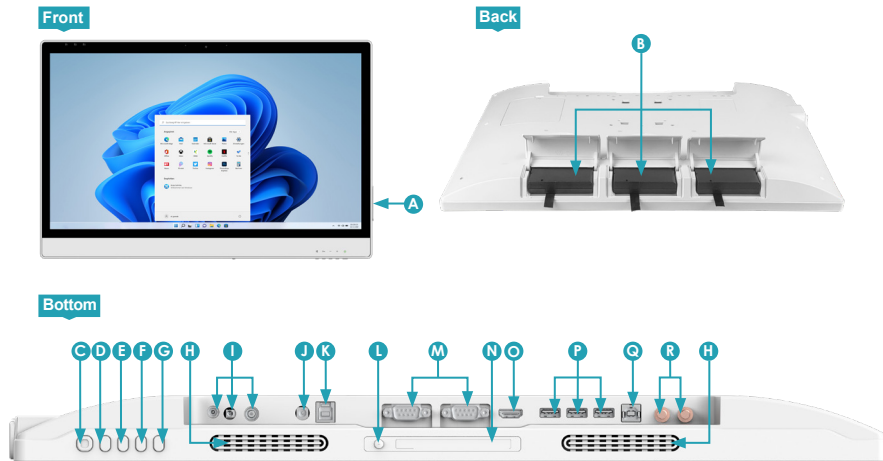

A USB 2.0	J DC-input
B Battery packs (optional)	K USB Type-B input
C Power button	L Reading light on/off button
D Right/Increment button	M COM ports
E Left/Decrement button	N Reading light
F Menu/Enter button	O HDMI input
G Exit/Return button	P USB 3.0
H Speakers	Q USB Type-B for touch screen connection
I DC-outputs	R Wi-Fi antennas

Cleaning the Screen

- A screen cleaning solution that is alcohol-free and non-abrasive can be used to clean the touch screen.
- Cleaning with a micro-fiber cloth is recommended.
- Please spread the solution onto the cloth and then clean the touch screen.

Cleaning the Anti-microbial Enclosure

- Use a soft/non-abrasive cloth moistened with water to clean the enclosure.
- To prevent scratching the anti-microbial coating, please wipe gently.

DT Research

Battery-Powered Medical Grade Monitor



BASIC OPERATION GUIDE

722M/724M

INTRODUCTION

Thank you for acquiring DT Research's Battery-Powered Medical Grade Monitor. With a 22-inch or 24-inch display and optional integrated batteries, the Battery-Powered Medical Grade Monitor offers optimal combinations of performance and power savings. With fully-integrated point-of-care modules within an elegant, space-saving design, the Battery-Powered Medical Grade Monitor is the optimum solution to enhance workflow and service, while pair with personal computers, embedded controllers, or thin clients.

Please take a few moments to review the contents of this document to ensure that the setup and startup proceed smoothly. The Battery-Powered Medical Grade Monitor is ready for use, out of the box, in its default configuration when powered by the power source provided. The following documentation offers guidance on the hardware elements and features of the monitor. Please refer to your device provider for information pertaining to the software operating system or software applications.

PACKAGE CONTENTS

- One AC-DC power adapter with power cord
- One HDMI cable
- One USB 2.0 Type-B to Type-A cable for the touch screen connection
- One USB 3.0 Type-B to Type-A cable for the system connection
- Two Wi-Fi connection cables
- Three DR202 Li-ion batteries (optional)
- Basic operation guide

PRECAUTIONS

- Always exercise care when operating and handling the Battery-Powered Medical Grade Monitor.
- Never disassemble any portion of the enclosure, as this will void any product warranty on the Battery-Powered Medical Grade Monitor.
- Do not use any AC/DC adapter other than the one provided with the device or a replacement acquired from the manufacturer.
- In the unlikely event that smoke, abnormal noise or strange odor is present, immediately power down the Battery-Powered Medical Grade Monitor and disconnect all power sources. Please report the problem to your device provider immediately.